

## Proclear® 1 day multifocal lens selection guide

# Different patients. One simple lens selection process.

### Easy lens selection from a unique approach.

- Centre-near aspheric design
- Designed with a single power profile
  - Preserves distance vision and limits visual compromise
- Near boost in the non-dominant eye—
  - Optimises near and intermediate vision and maintains binocularity
  - Accommodates patients up to +2.50 ADD
- Overall, a more natural visual experience

### Lens selection guide

**Step 1** Confirm refraction and determine eye dominance.

**Step 2** **Dominant eye:** Choose lens with appropriate distance Rx.  
**Non-dominant eye:** Choose lens with the recommended “near boost” power added to the distance Rx, per the patient’s ADD requirements:

Patient’s ADD	Near boost
up to +1.00	No near boost needed
+1.25 to +1.75	+0.75 D
+2.00 to +2.50	+0.75 D to +1.00 D

**Example refraction: -2.00 OU with +1.75 ADD**

**Proclear 1 day multifocal lens Rx**


Dominant eye	Non-dominant eye
-2.00	-1.25

**Step 3** Allow lenses to settle before assessing vision. If patient is satisfied with vision, dispense.

### Optimisation tips

To improve distance vision, perform over-refraction in +/- 0.25 D steps to dominant eye under binocular conditions. Many practitioners have found that adding +0.25 D to the dominant eye in Step 2 improves distance vision.

To improve near vision, perform over-refraction in +/- 0.25 D steps to non-dominant eye under binocular conditions.

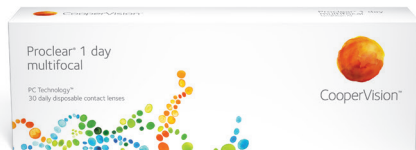

# One innovative approach. More reasons to choose Proclear.

## Improves near vision without disturbing distance vision.

- Ease of adaptation for lens wearers as they move through different stages of presbyopia
  - Designed with a single power profile
  - Less compromised distance vision
- Lens design contributes to a simplified lens-fitting experience
  - Reduces chair time and minimizes follow-up visits
- Broad, efficient parameter range
- Fresh approach to meet the vision needs of the 1-day wearer

## Different patients. One Proclear.

### Treats both eyes separately while maintaining binocularity

- Utilises different powers in each eye to improve near vision while minimising distance vision compromises
- Sends separate uncompromised signals to each eye, allowing the brain to produce a purer image

---

### A significant improvement over monovision

- Maintains binocular summation in higher ADDs to provide better visual performance than monovision
- Improves stereopsis, intermediate vision, and night vision
- True multifocal lens design

---

## Product specifications

### Proclear 1 day multifocal lens

Power range	+6.00 to -10.00 (0.25 steps; 0.50 steps after -6.00)
ADD power	Designed with a single power profile
Base curve	8.7 mm
Diameter	14.2 mm